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# USER MANUAL

## 1 • DESCRIPTION

The KINETEC *Adventa+* is a Continuous Passive Motion device for EXTENSION-FLEXION mobilization of the HIP and KNEE joints.

A - Lower leg support • B - Thigh support • C - Foot plate • D - Hand control • E - Power cord

## 2 • UNPACKING - PACKING

### Unpacking:

- a - Remove the KINETEC *Adventa+* from the packaging.
- b - Loosen the 2 black knobs (1) and adjust the thigh support in maximum position, then tighten the 2 knobs.
- c - Move the extension bars (2) to the self (locking position) by pulling the carriage away from the machine.
- d - Loosen the black knob (3) and adjust the foot plate.

The KINETEC *Adventa+* is ready to use.

### Packing:

- e - Loosen the 2 black knobs (1) and adjust the thigh support to the maximum position, then tighten the 2 knobs.
- f - Stop the unit at 0° (at the position indicated by the label).
- g - Pull the knob (4) and push the extension bars (2) back into the unit (while raising the knee axis).
- h-i - Adjust the thigh length to the minimum position and position the foot plate flat against the carriage.

a - Place the KINETEC *Adventa+* into the packaging.

The KINETEC *Adventa+* is ready to be shipped.

## 3 • ELECTRICAL CONNECTION: SAFETY FIRST.

Connect the power cord (5) to the KINETEC *Adventa* unit.  
(The Voltage should be between 100 and 240 Vac 50/60 Hz).

### IMPORTANT:

Check that the plug fits securely into outlet being used.

The outlet must be grounded.

Use only the power cord delivered with the unit.

Ensure that the power cord is not entangled with the machine before operating.

### Fuse change

Warning: Replace the fuse with one of the same type and value.

- in the input receptacle, 2 fuses T 500 mA 250V (6,3x32)  
- on the PCB, 1 fuse T 1.6 A (TR5)

## 4 • SAFETY

The healthcare professional determines the CPM protocol to be used (set up, duration and frequency of each session).

The patient must understand the Start/Stop/Reverse function of the hand control (see the paragraph 5).

The KINETEC *Adventa+* unit is CE marked (Directive 89/336/CEE, 93/42/CEE).

The KINETEC *Adventa+* does not emit magnetical, electrical or other types of interference or interfere with other devices.

**DANGER: Explosion risk if used with flammable anesthetics.**

## 5 • START UP

Turn on the power switch (6). The hand control will read **KINETEC ADVENTA+**

### Locking the hand control settings:

To lock the hand control control, simultaneously press the keys "-" and "+". The display will read "LOCK". When locked, it is possible to read the settings of the operational parameters and the START/STOP/REVERSE function will be operational.

To unlock the hand control, simultaneously press the "-" and "+" keys and the display will now read "UNLOCK".

We recommend that the switch be kept in the locked position during the session.

### Display

- 7 - 3 characters indicate the EXTENSION limit.
- 8 - 3 characters indicate the FLEXION limit.
- 9 - 8 characters indicate messages. (RUN, STOP, EXT, FLEX, SPEED, FORCE, PAUSE, etc.).
- 10 - 3 characters indicate the real angle of the KINETEC *Adventa+*.

### Key functions

- 11 - Reading or selecting EXTENSION limit from -3° to 110°.
- 12 - Reading or selecting FLEXION limit from 115° to 2°.
- 13 - Reading or selecting SPEED from 5 to 1.
- 14 - These keys will enable you to, by selecting, to reduce or increase the operating values (blinking value).
- 15 - This key enables you to start, stop and reverse the direction of movement.
- 15 - This key enables you to start, stop and reverse the direction of movement.

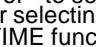
**IMPORTANT: for maximum safety, the hand control should always be given to the patient.**

16 - Reading or selecting PAUSE in EXTENSION or FLEXION limit from 900 seconds (15 minutes) to 0 second.

17 - Reading or selecting TIME COUNTER from 24H00MIN to 00H00MIN

At the end of the session, the display reads «TIMER 00H00MIN» and the unit stops at an average angle use.

Depress + or - to select a new session time.

18 - Reading or selecting movement inversion LOAD from:  = maxi load to  = mini load.

- RESET TIME function

First, stop the machine. Then, simultaneously depress the EXTENSION and FLEXION keys.

Press the START/STOP key twice to reset the counter.

This function will allow you to read the time of operation since the last reset operation was performed.

## 6 • SPECIFICATIONS

### Product

Weight: 11,2 Kg (26.4 lbs)

Device dimensions: 60x30x30cm  
(23.6x11.8x11.8 inches)

Range of motion: from -3° to 115°

Speed: from 50 to 150°/min for average femur length

Patient sizing: from 1.40m to 1.90m  
(4.5 to 6.25 feet)

### Electrical

Power supply:  
from 100 to 240 V~

Frequency: 50/60Hz

Electrical power: 50VA

Type B Class I device

### Environmental

- Storage / shipping conditions:

Ambient temperature: -40°C to +70°C.  
(-40°F to +160°F)

Relative humidity up to 90%.

- Operating conditions:

Ambient temperature: +10°C to +40°C.  
(+50°F to +105°F)

Relative humidity 80%.

## 7 • USE OF THE KINETEC PLASTIC SUPPORT

The KINETEC HYGIENIC pads are designed for rapid fitting, optimal hygiene and maximum patient comfort.  
 - To use and position the pads, please refer to drawing number 7.  
**WARNING:** place the notch cut into the pad at the knee joint.  
 For cleaning instructions, see paragraph 12.

## 8 • ADJUSTING FOR THE PATIENT

- Place the KINETEC *Adventa+* machine in a position that will be comfortable for the patient.
- Measure the length of the patient's femur (L); and adjust the thigh support to this measurement using knob (1).
- To install the patient on the KINETEC *Adventa+* machine, push the foot plate (C) up to the patient and tighten the knob (24).
- Secure the foot into the boot.

### IMPORTANT

Adjust the axis of the patient's knee (19) with the axis of rotation (20) of the KINETEC *Adventa+*.

## 9 • POSITIONING THE FOOT PLATE

The articulated foot plate allows the positioning of the foot in PLANTAR or DORSIFLEXION and the leg in EXTERNAL or INTERNAL ROTATION

- The knob (3) allows the foot to be positioned in plantar flexion or dorsiflexion.
- The knob (21) allows the leg to be positioned in external or internal rotation.
- The knob (22) allows the verticle adjustment of the foot.

## 10 • MAINTENANCE

After 2,000 hours of working time, the KINETEC *Adventa+* machine will need to be serviced (lubrication of the articulation joints, ball bearing nut and drive screw). The message SERVICE !! will appear on the hand control, when the unit is turned on, to indicate that the service interval has been reached.  
 You can continue use your KINETEC machine by depressing the START/STOP key. It is important that your KINETEC specialist is contacted to perform the maintenance operation.  
 At the end of the machines life, please return it to the manufacture, with all of the accesories for proper destruction.

## 11 • IF A BREAKDOWN

If a spare parts catalogue and/or service manual is needed, please request it from your KINETEC distributor.  
 If, after connecting the power cord to the KINETEC *Adventa+* machine and switching the machine ON (6),

- the display is blank:
  - Ensure that the wall power outlet is energized.
  - Check the fuses in the power receptacle (23) using fuses of the same type and value: 2 fuses T 500 mA 250V (6.3x32mm).
  - If the display does not indicate any information after following these steps, contact your KINETEC specialist.

If, after switching the power on to the KINETEC *Adventa+* using the switch (6) :

- your machine fails to operate and the display indicates 15 STOP 25 110, press the START/STOP key once again
- If your machine still fails to operate, contact your KINETEC specialist.
- If any of the following messages persist after switching the machine off and on again, and depressing the START/STOP key, contact your KINETEC specialist,
  - SERVICE D1 (failure of the copy potentiometer or cable)
  - SERVICE D2 (failure during motion)
  - SERVICE D3 (failure of motor direction or excessive current consumption),

## 12 • CLEANING

First, switch the CPM unit OFF and disconnect the power cord from wall outlet and the input receptacle.  
 Spray a DISINFECTANT product (i.e., PROPANOL/ISOPROPANOL or ALDEHYDE solution) on the SURFACES (leg support, plastic covers and metal parts).  
 For optimal hygiene, clean the pads after each patient, or all of the device's consumable pads may be thrown away.

## 13 • SYMBOLS

	TYPE B EQUIPMENT (insulated against electric shock)
	WARNING (consult accompanying documents)
	OFF (disconnected from the mains)
	ON (connected to the mains)
<b>Timer</b>	Timer

	SPEED
	EXTENSION
	FLEXION
<b>Pause</b>	Pause on extension or on flexion
<b>Force</b>	Reverse load

## 14 • GUARANTEE LIMITS

The KINETEC guarantee is strictly limited to free replacement or to repair of the part or parts acknowledged as defective, at the Smith & Nephew factory.  
 Smith & Nephew Kinetec guarantees its passive motion device against any manufacturing defect for 1 year from the date of purchase by the consumer.  
 Smith & Nephew Kinetec alone is qualified to judge whether the guarantee applies to the specific problem.  
 The guarantee cannot be applied if the apparatus has been subjected to abnormal use or has been used in circumstances other than those stipulated in our user manual.  
 The guarantee will not be applied in case of deterioration or accident due to negligence, lack of supervision or maintenance or an attempt to repair the equipment by unqualified personell.

## 15 • ACCESSORIES

- |                                  |                             |
|----------------------------------|-----------------------------|
| 15•1 Transport box with rolls    | 15•4 Foot support           |
| 15•2 Transport box without rolls | 15•5 Hand control protector |
| 15•3 Pads                        | 15•6 Bed sharp attachment   |

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We

**Smith & Nephew Kinetec**

(name of manufacturer or his representative established within the European Community)

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*declare on our own responsibility that product*

**KNEE KINETEC model ADVENTA+**

(brand name, modele,(N° of lot or series if necessary))

*complies with the assesement criteria of Council's Directives:*

- n° 93/42/CEE of June 14, 1993 - **MEDICAL DEVICES**  
(Annex II)  
mandatory date : June 14, 1998
- n° 89/336/CEE of May 3, 1989 - **ELECTROMAGNETIC COMPATIBILITY**  
mandatory date : January 1, 1996

*and the quality system of the manufacturer is in conformity with the following standard (s) :*

**EN 46001/96 - ISO 9001/94**

Certificate n° 0214/46001/9001 established by Gmed on 15/10/97

**Certificate for approval of full Quality Assurance System - Annexe II.3 -  
N° 0214/B2P3/1 established by Gmed**

*Additional information :*

**Product file : ADVENTA+ dated 17/12/98**

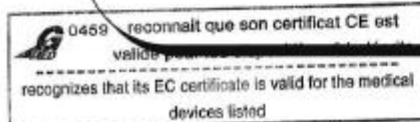
*Year in which CE mark was affixed : 1998*

**Tournes, December 17, 1998**  
(place and date)

**P. SONNET**  
General Manager

(Name, function and sign of authorized people)

23 DEC. 1998